

**Bruder®**

# THERMAL<sup>on</sup> SOLID STATE

Automatic Moist Heat Therapy Pad

## Setting New Standards for Safety & Performance

- Naturally Hydrating
- “Set and Forget” Control
- Solid State Reliability
- Heats Quickly
- Consistent Temperature
- 10-Preset Treatment Options
- Auto Shut-Off
- Five Year Warranty



*This UL-listed pad is ready whenever and wherever moist heat therapy is prescribed.*



**13" x 14" Heating Pad**  
with 1-Piece or 2-Piece Moist Heat Cover

**Deep Penetrating Moist Heat Relieves Pain and Soothes Aching Muscles & Joints**

# THERMALon

## SOLID STATE

### Moist Heat Therapy Pad

## Bringing a High Tech Solution to Moist Heat Treatments

### Consistent Therapy

The **Thermalon Solid State Heat Therapy Pad** is controlled by an electronic microprocessor. The time and temperature ranges available for selection have been programmed to comply with medically prescribed treatment parameters. The result is an effective therapeutic moist heat treatment every time the pad is activated and applied.

### Convenience

Naturally occurring moisture is supplied by the **moist heat therapy cover** - adding water is not necessary. Simply select the desired temperature level and treatment cycle and apply the **Thermalon Pad**. The **Thermalon Solid State Pad** conveniently delivers a deep-penetrating prescribed moist heat treatment every time.

### Safety

Precise time and temperature control, solid state heating element, natural hydration and built in fail-safe thermal fuses are all standard.


## SOLID STATE HEATING UNIT

Patented **Conductive Ink Technology** eliminates dangerous heater wires and thermostats. The pad heats quickly and provides consistent heat during treatment cycles.

Quadrant thermistor monitoring maintains selected heat levels across the entire pad surface.

Solid State design ensures reliable pad performance. Two fail-safe thermal fuses provide additional safety.

Pad is hermetically sealed and can be easily cleaned.

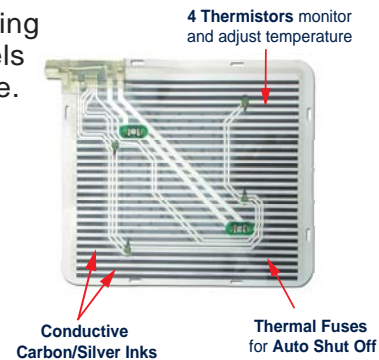

## MICROPROCESSOR CONTROL

The **Microprocessor** receives information from the thermistors and instantaneously responds to maintain the selected treatment settings.

The control offers 10 preset cycles and Auto Shut-Off for a full range of treatment options.




## NATURAL HYDRATION COVERS


The **Thermalon Solid State Therapy Pad** is available with two naturally hydrating moist heat cover options: **Model #53700** features a 1-piece cover while **Model #53400** features a 2-piece cover that is perfect for use in multi-user environments.

## Clinically Proven Pain Relief

**Thermalon Solid State Moist Heat Therapy Pads** take the guesswork out of delivering moist heat therapy whether at home or in the clinic. Heat therapy relaxes muscles and relieves joint pain and soreness by increasing blood flow to the injured area. An application of moist heat can also help to accelerate healing. Studies have proven that moist heat is more effective than dry heat for analgesia.

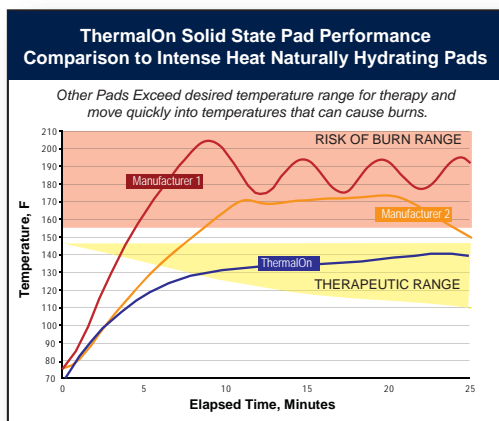

■ Manufacturer 1 Theratherm™ Sample Pad (166 Setting Shown)  
■ Manufacturer 2 Thermaphore™ Arthritis Pad (Full Cycle Shown)  
■ ThermalOn ThermalOn Solid State Pad (High Setting Shown)

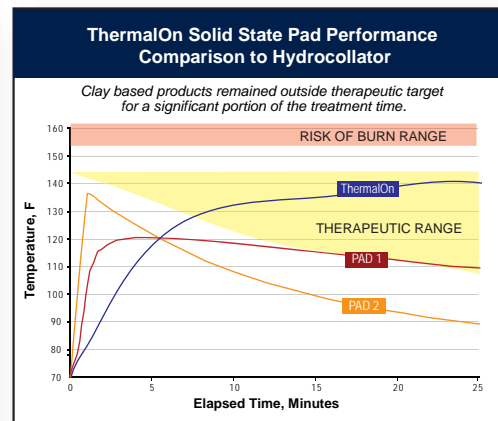

■ PAD 1 Hydrocollator Tested inside cover  
■ PAD 2 Hydrocollator Tested without cover  
■ ThermalOn ThermalOn Solid State Pad (High Setting Shown)